Strepsils (5)

SOOTHING THROAT SPRAY

INSTRUCTIONS FOR USE

Read the Instructions for Use carefully before using the product. Keep the Instructions for Use as you may need to read them again. If you have any questions, please consult your doctor or pharmacist.

HOW DOES STREPSILS SOOTHING THROAT SPRAY WORK?

Strepsils Soothing Throat Spray is a Self-Care Medical Device based on Polysac-Active[™] for the treatment of sore throat.

With Polysac-Active[™], which contains honey to help soothe a sore throat and

WHAT DO YOU NEED TO KNOW **BEFORE YOU USE STREPSILS SOOTHING THROAT SPRAY?**

Warnings and precautions

For oral use only

Consult your doctor or pharmacist if:

- Symptoms of sore throat do not improve within 5 days of treatment or worsen
- Persistent cough with lots of mucus production and/or high fever occur.
- Cough lasts longer than 3 weeks
- Prolonged hoarseness or voice problems occur
- · You have severe difficulties with swallowing
- You notice any unpleasant effects after using the spray

Do not use if:

- You are hypersensitive to any of the ingredients.
- You have a chronic cough (cough -lasting longer-than-3-weeks), chronicbronchitis, asthma, emphysema, chronic obstructive pulmonary disease – COPD – or large amounts of mucus are produced with the cough.

 Your persistent or recurrent cough is caused by heartburn or stomach complaints.

Strepsils Soothing Throat Spray in combination with other treatments or medicines

There are no known interactions with other treatments for sore throat or any other medicines, if used according to the Instructions for Use. However, it is advised to consult your doctor or pharmacist before combining the spray with other treatments.

WHO CAN USE STREPSILS SOOTHING THROAT SPRAY?

Strepsils Soothing Throat Spray can be used by adults and children from 3 years old. The spray always needs to bé administered by an adult or under supervision of an adult.

If you are pregnant or breastfeeding, or if you are suffering from diabetes, it is recommended to consult your doctor or pharmacist before using the spray.

HOW TO USE STREPSILS SOOTHING THROAT SPRAY?

Shake well before use. Use the nozzle on top of the bottle to spray directly towards the back of the throat. Spray 2 to 3 times every 3 hours or as required.

WHAT IS SORE THROAT AND DRY COUGH?

Sore throat can be a symptom of the common cold and flu. Symptoms of a sore throat include a painful throat, especially when swallowing, a dry, scratchy throat, redness in the back of the mouth, bad breath, a mild cough, swollen neck glands. A "dry cough" means it's tickly and doesn't produce any phlegm (thick mucus).

However, if your sore throat does not improve after a week, if you have had a cough for more than 3 weeks (persistent cough), if you get sore throats often, if you have a very high temperature, if you feel hot and shivery, if you have difficulty breathing, if you have a weakened immune system – for example, because of diabetes or chemotherapy, if you have chest pains, unexplained weight loss, or if you are worried about your sore throat you should see your doctor. A sore throat and a dry cough can sometimes be caused by serious underlying conditions. If you have any concerns, you should consult your doctor or pharmacist.

WHAT ARE THE POSSIBLE SIDE **EFFECTS OF STREPSILS SOOTHING THROAT SPRAY?**

There are no known side effects when used according to the Instructions for Use.

WHAT ARE THE INGREDIENTS OF **STREPSILS SOOTHING THROAT SPRAY?**

Polysac-Active™ (contains honey), Aqua-(water), Sodium Chloride, Laureth-9, Potassium Sorbate, Chondrus Crispus Extract, Peppermint Oil, Zingiber Officinale Root Oil, Limonene, Citral, Linalool, Geraniol, Citric Acid.

HOW TO STORE STREPSILS SOOTHING THROAT SPRAY?

Keep in the original packaging at room temperature (15-25 °C).

Keep out of sight and reach of children.

Do not use after the expiry date as printed on the spray applicator and outer box.

After first opening, discard after 6 months.

WHAT IS THE CONTENT OF THE PACKAGE?

The package contains a 20 mL e spray bottle and enclosed Instructions for Use.

Manufacturer: Medical Brands Developments B.V., Piet Heinkade 199, 1019 HC Amsterdam, the Netherlands.

Distributor: RB UK Commercial Ltd, 103-105 Bath Road, Slough SL1 3UH, UK

Reckitt Benckiser Ireland Limited, Citywest Business Park, Dublin 24, Ireland

This Instruction for Use was last revised in May 2019

MEANING OF SYMBOLS USED

		Store this product out of sight and reach of children
	15°C	The product must be stored in a safe, dry and dark place at room temperature (15-25°C)
_	LOT	Batch number of the product
	\triangle	Special attention must be brought to warnings
	6M	Period-after-opening symbol





RB Artwork and Print Specification

Artwork Type: IDM Commercial 3101904 ished Goods Co d Party Code 20TS RB UK L00

Guides 2 (if applicable)

230

200

19

160

150

110

-8

-8

-2

-9

20

-6

30

BARCODE INFO





UNDER NO CIRCUMSTANCES SHOULD THIS ARTWORK BE ALTERED WITHOUT PRIOR PERMISSION FROM TRIDENT.

G=2: O=0: R=0: - MW - 21/05/19 11:02:56

CUSTOMER INFO:

Minimum Point Size = 8.32pt

Production line				
Manufacturing site				
Pack Tech approver	1 st	2 nd],
Date				
Check Criteria	Details & Commer	nts	Che	
Approvals			1st	2 nd
Market Approval				
Regulatory Approval				
3rd Party Approval				
Technical Drawing/Cutter ref no.				
PPI component code correct on artwork?				
Correct drawing for the production line?				_
Profile shape & dimensions Graphics layout/orientation				
Unwind diagram (PPTD1022/1-20)				
Printable areas/bleed				
Varnish free areas				
Variable coding position & dimensions				-
Varnish and print requirements				-
Pre-printed headers (alignment and suitabilit	tv)			-
Embossing, Braille & Foil Blocking	.,,			-
Separations on artwork			- 	
Alignment within cutter				
Braille structure				-+
Verification Code Type; Pharma Code / 2D / 0	CV			
Bar sequence and/or number correct Position				
Size (height, width, spacing, font)				
Direction of read				
Light margins and print free areas				
Colour				
Repeat distance (reeled material)				
Cross check related docs ie BOM				
Bar code Type; EAN, ITF, Code39, etc				
Magnification / width and height				
Colour (e.g. black print on white background))			
Light margins & indicators				
Bearer bars & H gauges (if required)				
Symbol grade (Pass = A,B,C; Action = D,F)				
Bar code number (human readable)				
Bar code number (encoded & hidden data)				
Confirm bar code number from source				
Cross check related docs (JDE/PID)				
Photo-electric cell mark / edge marks				
Present & per Tech drawing?				
Correct pitch?				
Additional markings				
e-mark, recycle logo etc				
Legend box (RB specification)				
Correct component code				
Correct D spec / has cutter changed?				
Correct printer / has printer changed? Correct Substrate				
A/W is set up appropriately for this substrate Edge mark details				—[
Pharmacode details				
Colours & print process				
Varnish Parada information				
Barcode information				
General details are correct				
Pass / Fail comments and Signature				

Version 003 19/9/2016

RB Artwork and Print Specification

240 250

230

210

160

___8

___8

70

—9

29

-64

30

-20

__ mm 10

Ш	Trident Reference No:	RB375123
	ZEN Ref:	TR1614854
	Action:	С
$\vdash\vdash$	Brand:	Strepsils
	Category:	Throatcare
	Segment Group:	Soothing Throat Spray
	Segment:	25ml
	Pack Size:	25ml
	Market/Country:	UK
	Date:	21/05/19
-	RBH Contact:	Maggie Du
	Artwork Type	: IDM Commercial
	Component Code (2D if app	
-	Parent Technical Packagin	g
	Specification:	D8360351
	Finished Goods Code:	3101904
-	Supply Point:	3rd Party Medical Brands
	3rd Party Code	20TS.RB.UK.I.00
	Pharmacode No/NE:	N/A
-	Edgemark Position:	N/A
	CAD Cam Ref:	S-Lft-D8360351-297mm x 210mm
	Printer:	Dobber Healtcare Leaflets
	Substrate:	Paper White
-	Tankai and R Nian Delastica d	
	Technical & Non Printing I	
	Cutter	Cutter 2 (if applicable)
-	Guides	Guides 2 (if applicable)
	0 1 " " "	
	Colours (Leaflet)	
	Process Black	
\perp	LTH	
-		
-		
-		
-	BARCOL	E INIEO
Ш	DAKCOL	'E IINFO
Н	Barcode Type:	N/A
Ш	Barcode Number:	N/A
	Magnification:	N/A
\vdash	Truncated By:	N/A
Щ	Full Height:	N/A
1 1	Bar Height (Smallest Bar):	N/A



I: +44 (U) 1402 OZDIOO use note that any low resolution paper Canon colour copies ciated with this job should be referred to for content, layout and colour separation only.

UNDER NO CIRCUMSTANCES SHOULD THIS ARTWORK BE ALTERED WITHOUT PRIOR PERMISSION FROM TRIDENT.

Sm•Art check results: G=2; O=0; R=0; - MW - 21/05/19 11:02:56