

# Earol<sup>®</sup>

## OLIVE OIL SPRAY

Earol<sup>®</sup> spray is a simple and effective method of softening or naturally removing ear wax and thus reducing the need for syringing, or to facilitate syringing should it prove necessary.

Earol<sup>®</sup> spray has a simple yet unique delivery system capable of delivering a discrete unit dose of Olive Oil into the outer ear, enhancing ease of use, reducing over application and significantly reducing spillage/mess. Please read this leaflet carefully before you use Earol<sup>®</sup> spray.

### BEFORE USING A NEW BOTTLE OF EAROL<sup>®</sup> SPRAY

■ Before using a new bottle of Earol<sup>®</sup> spray, warm in palm of hand, shake well, then press the actuator down 3 or 4 times until you get a spray (See Diagram b). When doing this do not point the actuator at yourself or anyone else. This procedure will prime the system and ensure the product is ready to use.

### DIRECTIONS FOR USE

- Earol<sup>®</sup> is most effective when warmed in palm of hand and shaken well before use.
- Apply Earol<sup>®</sup> as directed by your Doctor or nurse.
- Adults or children over 12 years of age require 1 or 2 applications of Earol<sup>®</sup> twice daily for 4 or 5 days before consultation for wax removal.
- Thereafter your Doctor or nurse may suggest using Earol<sup>®</sup> once a week as a preventative measure.
- Pull the upper ear or Pinna upwards and backwards (See Diagram a)
- Holding the bottle in an upright position with the nozzle placed in the ear canal depress the actuator fully 1 or 2 times (See Diagram b)
- Massage gently the Tragus just in front of the ear (See Diagram a)
- Repeat the procedure with the other ear if necessary.
- The actuator button can be removed and rinsed in hot water after use and then seated back on the stem.

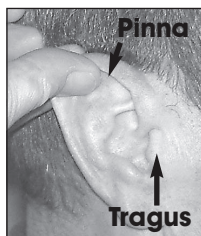


Diagram a



Diagram b

# Earol®

## OLIVE OIL SPRAY

### ADDITIONAL INFORMATION

- Patients who have a perforated ear drum or ear pain should only use the product under medical supervision.
- Earol® spray should not be used for children under one year of age without medical advice.
- Olive Oil is a natural product and has wide daily applications, as such no side-effects have been reported with this product or with this product interacting with other medicines. However this product should not be used if you have a sensitivity or allergy to Olive Oil.
- There are no known harmful side-effects when using this product during pregnancy and/or breast feeding, however you should always inform your Doctor before use.
- There are no known side-effects from over or under dosing with this product.
- If you feel you have experienced an allergic reaction stop using this product and inform your Doctor or pharmacy immediately.

### WARNINGS

- Earol® is for **single patient use only**.
- Always store Earol® spray in an upright position at normal room temperature.
- The product should be discarded three months after first daily application.
- Dispose of the product as per Local Authority advice.
- Do not spray near naked flame.
- Remember to keep all medicines in a safe place where children cannot reach them.
- Earol® is only for use in the ear canal. Do not spray near eyes, nose or mouth.

**Earol® Olive Oil spray is manufactured in the UK by HL Healthcare Ltd,  
10 Hazelwood Road, Hale, Cheshire WA15 9AX.**

Earol® is a registered trademark of HL Healthcare.  
Leaflet produced September 2007. Ref: GK002